



Sunflower Warmer

Radiant Warmer



Convenient design.

Sunflower Warmer
Hi-Low/Tilt



HL Stand	✓
Fixed Stand	—
Apgar Timer	✓
CPR Timer	✓
Mattress platform tilting	✓
Rotary damper	✓
Drawer	▲
Cabinet	—
Double-wheel Caster	✓
Single-wheel Caster	—
X-Ray cassette tray port	✓
Examination light	✓

✓ : Included as standard
— : Not available ▲ : Optional

Sunflower Warmer
Cabinet/Tilt




HL Stand	—
Fixed Stand	✓
Apgar Timer	✓
CPR Timer	✓
Mattress platform tilting	✓
Rotary damper	✓
Drawer	—
Cabinet	✓
Double-wheel Caster	✓
Single-wheel Caster	—
X-Ray cassette tray port	✓
Examination light	✓

✓ : Included as standard — : Not available

Versatile design.

Sunflower Warmer **Stand**

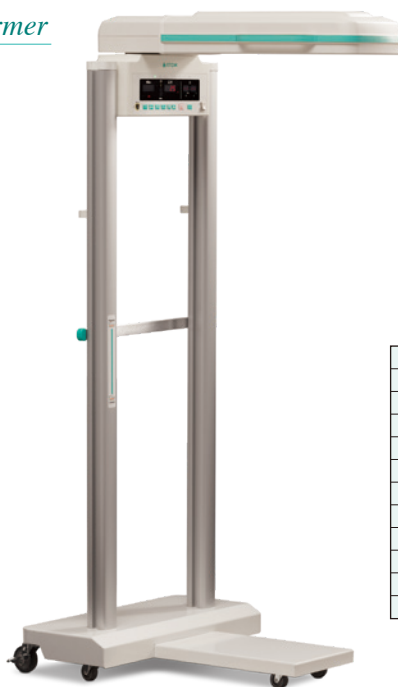

HL Stand	—
Fixed Stand	✓
Apgar Timer	✓
CPR Timer	✓
Mattress platform tilting	—
Rotary damper	—
Drawer	—
Cabinet	—
Double-wheel Caster	—
Single-wheel Caster	✓
X-Ray cassette tray port	—
Examination light*	✓

✓ : Included as standard — : Not available
* Except for US

Space-saving design.

Sunflower Warmer **Wall Mount**



HL Stand	—
Fixed Stand	—
Apgar Timer	✓
CPR Timer	✓
Mattress platform tilting	—
Rotary damper	—
Drawer	—
Cabinet	—
Double-wheel Caster	—
Single-wheel Caster	—
X-Ray cassette tray port	—
Examination light*	✓

✓ : Included as standard — : Not available
* Except for US

* Load capacity of the wall permitting.



for use in
NICU

Optional accessories for neonatal care can be mounted.



Hi-Low/Tilt

* The drawer shown is optional


Surgical operations can be performed in the NICU.

◆ Accessible workspace for surgical treatment

- A range of peripheral devices such as our resuscitation unit (T-piece resuscitator, oxygen blender and suction unit), phototherapy unit, examination light, weight monitor etc. can be mounted through F-Rail.

“Baby-friendly” function.

◆ Support easy access

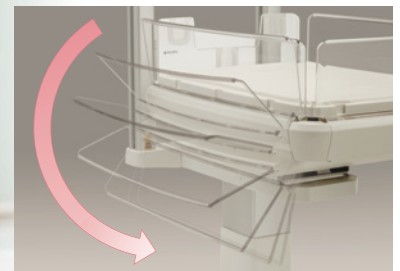

- Baby-guard with rotary damper that moves slowly.



for
Obstetric
use

For urgent postnatal treatment.




Hi-Low/Tilt

* The drawer shown is optional

Prompt resuscitation can be performed.

◆ **Resuscitation treatment in compliance with ILCOR Consensus 2020 can be achieved.**

- CPR Timer prompts the user to assess the health condition of the infant every 30 seconds.

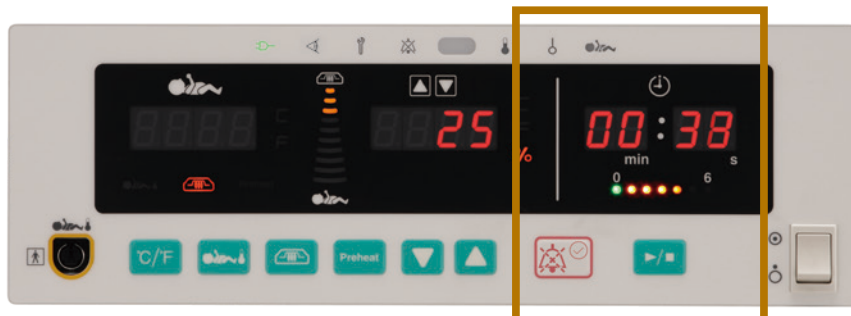

* The image shows the display for regions with a 230 V power specification. For regions with a 120 V power specification, the display differs slightly.

- Resuscitation unit (T-piece resuscitator, oxygen blender and suction unit) and weight monitor, etc. can be mounted to F-Rail.



Products Lineup

Sunflower Warmer

Type	Atom's code	Description
Hi-Low/Tilt	61564	AC120V±10%
	61565	AC230V±10%
Cabinet/Tilt	61566	AC120V±10%
	61567	AC230V±10%

Type	Atom's code	Description
Stand	61572	AC120V±10%
	61573	AC230V±10%
Wall Mount	61574	AC120V±10%
	61575	AC230V±10%

Specifications


Power requirements	Customer specified	
Power consumption	Main Unit : 700VA HL Stand : 120VA	
Frequency	50/60Hz	
Type of protection	Class I	
Degree of protection	Type BF applied part	
Heater capacity	500W	
Dimensions	HL/Tilt : 640(W)x1140(D)x1770~1970(H) mm Cabinet/Tilt : 610(W)x1060(D)x1870(H) mm Stand : 730(W)x950(D)x1840(H) mm Wall mount : 450(W)x950(D)x300(H) mm	
Height to mattress	HL/Tilt	800~1000 mm
	Cabinet/Tilt	900 mm
	Stand	660~860 mm
	Wall mount	Depends on the wall mount height
Weight	HL/Tilt	Approx. 86kg
	Cabinet/Tilt	Approx. 93kg
	Stand	Approx. 51kg
	Wall mount	Approx. 20kg

Accessories	HL/Tilt	Skin temp. probe....1, Dust cover....1, Cord holder....2, Operation Manual....1
	Cabinet/Tilt	Skin temp. probe....1, Dust cover....1, Cord holder....2, Operation Manual....1
	Stand	Skin temp. probe....1, Cord holder....2, Operation Manual....1
	Wall mount	Skin temp. probe....1, Operation Manual....1
Heater output setting range		0~100% (in 5% increments)
Heater output indication		In 10 levels
Skin temp. setting range		34.0~38.0°C / 93.2~100.4°F (in 0.1°C/°F increments)
Skin temp. display range		30.0~42.0°C / 86.0~107.6°F
Skin temp. display accuracy		±0.3°C / ±0.5°F
Timer		0~60 minutes 00 second (in one second increments) Apgar mode / CPR mode selectable
Mattress platform tilting		Longitudinal tilt of ±13° (freely adjustable)
Alarms		Set temp./ heat, Baby check, Power failure, System failure, Skin temp. probe, Canopy tilt


Dimensional drawing

Unit:mm


■ Hi-Low/Tilt

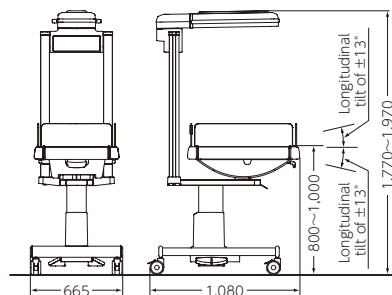



■ Cabinet/Tilt

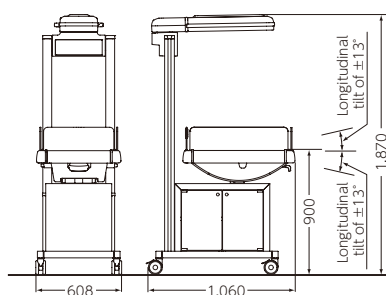

■ Stand

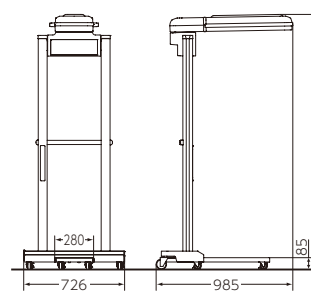

■ Wall Mount

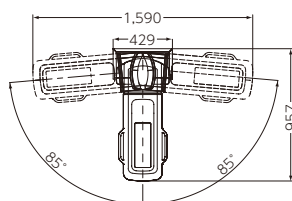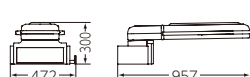

Atom Medical products are produced under strict production control in Japan.
We provide value and trust of the highest quality for the tiny baby's lives in the world.

ATOM MEDICAL CORPORATION

<https://atomed-global.com>